

Prontosan™ Wound Gel

For maceration of wound coatings and cleaning of wounds and for moistening and lubricating absorbent wound dressings

STERILE
Aseptic Fill < 1 CFU/mL
NONPYROGENIC
fluid and fluid path prior to
removal of cap.



Single use solution in
multiple use container.
Do not resterilize.



See instructions
for use.



Keep away from
sunlight.



Expiration Date

Rx only



Lot Number

Pictorials are for reference only.



Reorder Number




Latex Free



DEHP Free

880318 P-7616-1 REV. 3/09

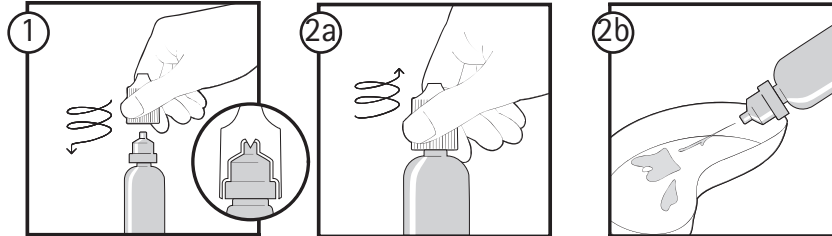

Description:

Prontosan Wound Gel cleanses the tissue surface, even when surfaces are difficult to access such as deep wound cavities, fissures, and wound pockets.

Appearance:

Clear, colorless, and virtually odorless, aqueous solution.

Summary / technical information:

Prontosan Wound Gel contains:

Glycerol,
Hydroxyethylcellulose,
0.1%
Undecylenamidopropyl
Betaine,
0.1% Polyaminopropyl
Biguanide (Polyhexanide),
Purified Water.

Indications:

Prontosan Wound Gel is intended for moistening and cleansing wounds by maceration of wound coatings and for moistening and lubricating absorbent wound dressings for ulcers, burns, post-surgical wounds and abrasions.

Prontosan Wound Gel may be used:

- For cleansing wounds by maceration of wound coatings
- For cleansing and moistening chronic skin wounds
- For moistening of bandages and wound dressings, such as compresses, gauze, pads, sponges, etc.
- During dressing changes to loosen encrusted bandaging or other encrusted wound dressings.

Side effects:

There are no known side effects.

Contraindications:

- Prontosan Wound Gel should not be used if there is a history of allergy to any of the ingredients.
- Do not use in the presence of hyaline cartilage.
- Do not use in combination with anionic tensides as these may impair preservation.
- Mixing Prontosan Wound Gel with other wound cleansing soaps, ointments, oils,

or enzymes may lower efficacy. When such substances need to be removed from a wound, ensure the entire wound area is thoroughly rinsed with a cleansing solution such as Prontosan Wound Irrigation Solution.

Warnings:

- For external use only on skin wounds.
- Do not use for infusion or injection.
- Do not ingest.

Shelf-life:

Store at room temperature. Avoid excessive heat. Protect from freezing. Use prior to expiration date noted on the package label.

Shelf-life after opening:

28 days.

Preservation and shelf-life:

Although no longer sterile once the bottle has been opened, because of preservative, there is a possible shelf-life of 28 days after opening the bottle if it is closed immediately after use and the top of

the bottle is protected from contamination or direct contact.

Instructions For Use

For general use:

Rinsing and cleansing the wound and surrounding area is recommended prior to application of Prontosan gel. Prontosan Wound Irrigation Solution can be used for this purpose. Refer to cleansing solution instructions for use.

Apply a 3-5 mm coating of Prontosan Wound Gel directly to shallow flat wound surfaces or deep wound fissures.

Bandages or dressings soaked in Prontosan Wound Gel can be used for cleaning as required.

Prontosan Wound Gel can be used to sustain wound moisture for the duration of time between dressing changes per institutional protocol.

Note: Prontosan Wound Gel can be warmed to body temperature immediately prior to use.

Directions: Use Aseptic Technique.

1. To open, screw cap down fully to pierce container.
2. Remove cap and proceed with use as required in accordance with institutional protocol.
3. Replace cap after use in accordance with institutional protocol.

B | BRAUN

Manufactured for:
B. Braun Medical Inc.
Bethlehem, PA 18018-3524
1-800-227-2862
www.bbraun.com