

Prontosan™ Wound Irrigation Solution

For cleaning wounds and for moistening and lubricating absorbent wound dressings

STERILE
Aseptic Fill < 1 CFU/mL
NONPYROGENIC
fluid and fluid path prior to
removal of cap.



Single use solution in
multiple use container.
Do not resterilize.



See instructions
for use.



Keep away from
sunlight.



Expiration Date



Lot Number



Reorder Number



Latex Free




DEHP Free

Rx only

Pictorials are for reference only.

880287 P-7579-1 REV. 6/08

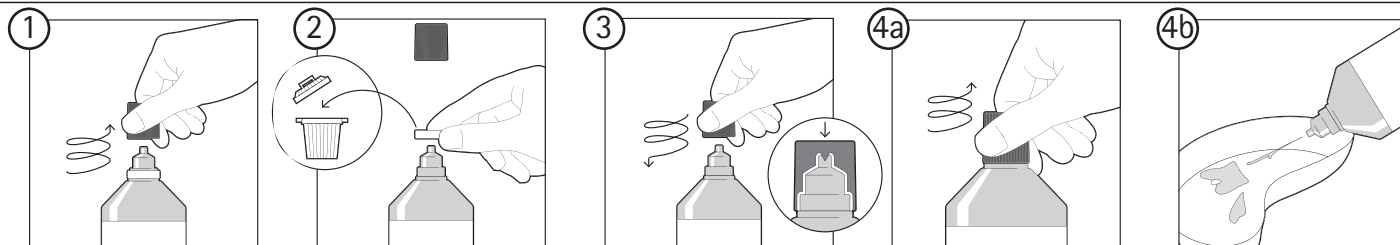

Description:

Prontosan Wound Irrigation Solution cleanses the tissue surface, even when surfaces are difficult to access such as rhagades, fissures and wound pockets.

Appearance:

Clear, colorless and virtually odorless, aqueous solution.

Summary / technical information:

Prontosan Wound Irrigation Solution contains:
Purified Water,
0.1 % Undecylenamidopropyl Betaine,
0.1 % Polyaminopropyl Biguanide (Polyhexanide).

Indications:

Prontosan Wound Irrigation Solution is intended for cleaning wounds and for moistening and lubricating absorbent wound dressings for ulcers, burns, post-surgical wounds, and abrasions. The application of the solution across the wound provides for the mechanism of action and aids in the removal of foreign objects such as dirt and debris.

Prontosan Wound Irrigation Solution may be used:

- for cleansing and moistening chronic skin wounds
- for moistening of bandages and wound dressings, such as compresses, gauze, pads,

sponges, gels, hydrofibres, alginates, hydrocolloids etc.

- during dressing changes to loosen encrusted bandaging or other encrusted wound dressings.

Side effects:

There are no known side effects.

Contraindications:

- Prontosan Wound Irrigation Solution should not be used if there is a history of allergy to any of the ingredients.
- Do not use in the presence of hyaline cartilage.
- Do not use in combination with anionic tensides as these may impair preservation.
- Mixing Prontosan Wound Irrigation Solution with other wound cleansing soaps, ointments, oils, or enzymes may lower efficacy. When such substances need to be removed from a wound, ensure the entire wound area is thoroughly rinsed with Prontosan Wound Irrigation Solution.

Warnings:

- For external use only on skin wounds.
- Do not use for infusion or injection.
- Do not ingest.
- Bottles that become contaminated or have come into contact with the wound must be disposed of after use.

Shelf-life:

Store at room temperature. Avoid excessive heat. Protect from freezing. Use prior to expiration date noted on the package label.

Shelf-life after opening:

28 days.

Preservation and shelf-life:

Although no longer sterile once the bottle has been opened, because of preservative, there is a possible shelf-life of 28 days after opening the bottle if it is closed immediately after use and the top of the bottle is protected from contamination or direct contact.

Instructions For Use

For general use:

Cleanse the wounds with Prontosan Wound Irrigation Solution before further treatment is carried out.

Bandages or dressings soaked in Prontosan Wound Irrigation Solution can be used for cleaning as required.

Cleansing the wound site:

Massive cleansing of a large area around the whole wound site with Prontosan Wound Irrigation Solution is recommended.

When required, wash and decontaminate the whole section of the body or the whole body of the person affected according to institutional protocol.

For wounds covering a large area and wounds difficult to access, bathe the whole section of the body with Prontosan Wound Irrigation Solution, for at least 15 minutes. The solution should be used in an undiluted form.

Note: Prontosan Wound Irrigation Solution can be warmed to body temperature immediately prior to use.

For use in the case of encrusted bandages or when changing dressings is problematic:

In cases where bandages are difficult to release, wetting the wound dressing with Prontosan Wound Irrigation Solution is advisable until the bandages can be gently released without traumatizing the surface of the wound. If stubborn, large encrustations are present, the whole section of the body with the dressing can be saturated or bathed in Prontosan Wound Irrigation Solution until the dressing can be easily released.

Directions: Use Aseptic Technique.

1. Remove cap.
2. Remove and discard white ring below cap.
3. Replace cap. Ensure cap is screwed down fully to pierce container.
4. Remove cap and proceed with use as required in accordance with institutional protocol.
5. Replace cap after use in accordance with institutional protocol.
6. Record date opened on bottle.
7. Solution may be used for up to 28 days.
8. Discard any unused solution and bottle.

B | BRAUN

Manufactured for:
B. Braun Medical Inc.
Bethlehem, PA 18018-3524
1-800-227-2862
www.bbraun.com