

Prontosan™ Wound Irrigation Solution

For cleaning wounds and for moistening and lubricating absorbent wound dressings

STERILE
Aseptic Fill < 1 CFU/mL
NONPYROGENIC
fluid and fluid path prior to
removal of cap.



For single use only.
Do not resterilize.



See instructions
for use.



Keep away from
sunlight.



Expiration Date



Lot Number



Reorder Number



Latex Free



DEHP Free

Rx only

880256 P-7249-1 REV. 6/08

Description:

Prontosan Wound Irrigation Solution cleanses the tissue surface, even when surfaces are difficult to access such as rhagades, fissures and wound pockets.

Appearance:

Clear, colorless and virtually odorless, aqueous solution.

Summary / technical information:

Prontosan Wound Irrigation Solution contains:
Purified Water, 0.1 %
Undecylenamidopropyl Betaine, 0.1 %
Polyaminopropyl Biguanide (Polyhexanide).

Indications:

Prontosan Wound Irrigation Solution is intended for cleaning wounds and for moistening and lubricating absorbent wound dressings for ulcers, burns, post-surgical wounds, and abrasions. The application of the solution across the wound provides for the mechanism of action and aids in the removal of foreign objects such as dirt and debris.

Prontosan Wound Irrigation Solution may be used:

- for cleansing and moistening chronic skin wounds
- for moistening of bandages and wound dressings, such as compresses, gauze, pads, sponges, gels, hydrofibres, alginates, hydrocolloids etc.
- during dressing changes to loosen encrusted bandaging or other encrusted wound dressings.

Side effects:

There are no known side effects.

Contraindications:

- Prontosan Wound Irrigation Solution should not be used if there is a history of allergy to any of the ingredients.
- Do not use in the presence of hyaline cartilage.
- Do not use in combination with anionic tensides as these may impair preservation.
- Mixing Prontosan Wound Irrigation Solution with other wound cleansing soaps, ointments, oils, or enzymes may lower efficacy. When such substances need

to be removed from a wound, ensure the entire wound area is thoroughly rinsed with Prontosan Wound Irrigation Solution.

Warnings:

- For external use only on skin wounds.
- Do not use for infusion or injection.
- Do not ingest.

Shelf-life:

Avoid excessive heat. Protect from freezing. Store at room temperature. Use prior to expiration date noted on the package label.

Instructions for use

For general use:

Cleanse the wounds with Prontosan Wound Irrigation Solution before further treatment is carried out.

Bandages or dressings soaked in Prontosan Wound Irrigation Solution can be used for cleaning as required.

Cleansing the wound site:

Massive cleansing of a large area around the whole wound site with Prontosan Wound

Irrigation Solution is recommended.

When required, wash and decontaminate the whole section of the body or the whole body of the person affected according to institutional protocol.

For wounds covering a large area and wounds difficult to access, bathe the whole section of the body with Prontosan Wound Irrigation Solution, for at least 15 minutes. The solution should be used in an undiluted form.

Note: Prontosan Wound Irrigation Solution can be warmed to body temperature immediately prior to use.

For use in the case of encrusted bandages or when changing dressings is problematic:

In cases where bandages are difficult to release, wetting the wound dressing with Prontosan Wound Irrigation Solution is advisable until the bandages can be gently released without traumatizing the surface of the wound. If stubborn, large

encrustations are present, the whole section of the body with the dressing can be saturated or bathed in Prontosan Wound Irrigation Solution until the dressing can be easily released.

Directions: Use Aseptic Technique.

1. To open container, firmly grasp tab between thumb and forefinger. Twist until tab separates from container.
2. Proceed with use as required in accordance with institutional protocol.
3. Discard any unused solution and container.

B | BRAUN

Manufactured for:
B. Braun Medical Inc.
Bethlehem, PA 18018-3524
1-800-227-2862
www.bbraun.com